Omikron CallHelp 500RP Handleiding Versie: 1.09

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Omikron CallHelp 500RP Manual Versie: 1.09

<u>Omikron CallHelp 500RP Repeater*</u>	<u>Conformity Statement</u>
<p>Technical information:</p> <p>Voltage adapter: 12V/1A Frequency: UHF 433.92Mhz. RF Power: 10Mw Standby: <40 hour ** Back-up battery: Lithium battery 3.7V 800Mah Standby current: 20Ma Transmitting current: 90Ma Charging time: 50 hour</p> <p>*For use with CallHelp Paging systems or Buttons with Alpha Numeric Tekst Messages.</p> <p>**Depending on usage.</p> <p>Non Waterproof. Warranty 1 year. Except damage from falling, when opened cabinet, modification, wrong use, broken screen.</p> <p>***We advise to replace every year the backup battery for reliable working.</p>	<p>Herewith we declare that the following product:</p> <p>Omikron CallHelp 500RP Repeater is meeting all EU requirements in accordance Directive 2014/53/EU (RED)</p> <p>Product description: HF oproepzender ID Markering: 500RP Class 1 Equipment</p> <p>In accordance: 2014/53/EU European Radio Equipment Directive (RED) 2011/65/EU Restriction of Hazardous Substances (RoHS) and that the following harmonized European standards are applied: Health and safety: EN 60 950-1:2006 + A11:2009 + A1:2010 + A12:2011 + AC:2011 + A2:2013 Electromagnetic compatibility: EN 301 489-1 : V2.1.1 EN 301 489-3 : V1.6.1 Use of the radio frequency spectrum: EN 300 220-2 : V3.1.1 Assessment of hazardous substances: EN 50581:2012</p>  <p>Omikron Europe Theo Kwantenstraat 53 3223CE Hellevoetsluis The Netherlands tel. 00 31 (0) 642705689 web:http://www.omikron-europe.com e-mail : info@omikron-europe.com</p> <p>Hellevoetsluis, 1.10.2019</p>  <p>P. Konstapel CEO</p>

System Setup

Charge fully before use. Turn on with on/off switch.

- 1.6** Long press SET switch side until LED RED quickly activates PC Setup Mode.
- 1.7** PC Setup mode, see manual PC software. Escape program setup pressing long SET or after 60 seconds automatically.
- 1.8** Press again short activates System Coding by 500TXxBTTN. LED flashes fast GREEN. Press a 500TXxBTTN. LED Flashes 1x GREEN. Escape program setup pressing long SET or after 60 seconds automatically.
- 1.9** Press again short activates Range TEST mode. LED flashes slowly GREEN. Activate Range Test on a 500TX3BTTN or activate Range Test on a 400TX. The repeater will retransmit every received signal 1X. Test Range with 500RX in TEST mode. Escape program setup pressing long SET or after 60 seconds automatically.
- 1.10** Press again short activates automatic RANGE TEST mode. LED flashes slowly RED. During 20 minutes every 1 second a transmission is sent. Test with 500RX or next 500RP in test mode the range. Escape program setup pressing long SET or after 60 seconds automatically.

With PC software you can setup the number of the repeater, automatic hourly control and systemcode.

Repeaters can be numbered sequential. System will test every hour if all repeaters are present. This will be visual on display of 400TX or in 500RX pager.

If any repeater receives a signal transmission from any button in its range it will retransmit by all repeaters. Placement of repeaters is preferably sequential.